

Directorate of Pathology  
Department of Blood Science

BS/FT 94

Revision 001

DATE 02/01/2013

ROTEM®: Competency assessment.			
Name:	Demonstrated	Performed	Date
<b>Important: Whole blood citrated sample (Green Top), unspun.</b>	Acknowledged		
Knows where reagents and consumables stored			
Sample to be kept warm in the holding area			
Select appropriate reagents: Ex-Tem (red), In-Tem (blue), Fib-Tem (brown)			
Able to log onto system			
Able to select appropriate channel and test.			
Is able to set up reagents & follow on screen instructions			
Input patient data onto system. Sample time is inserted into comments box.			
Copy and paste this information to the other channels			
Able to set up channels: Install cups and pins.			
Start appropriate channel and initiate pipetting program (test reagent prep and pipetting).			
Insert cup/holder onto probe/pin and start analysis for appropriate channel			
Once appropriate channels started – select 4-TEM module (displays all 4 graphs). Display standard overlay.			
Once analysis completed – tap any channel to return to individual channels. Stop and clear channels			
Print appropriate results.			
Discard cups and pins.			
Logout and quit the program.			